


# DRUGFORMATION.COM

## Instructions for Single and Multi Drug Screen Test Card

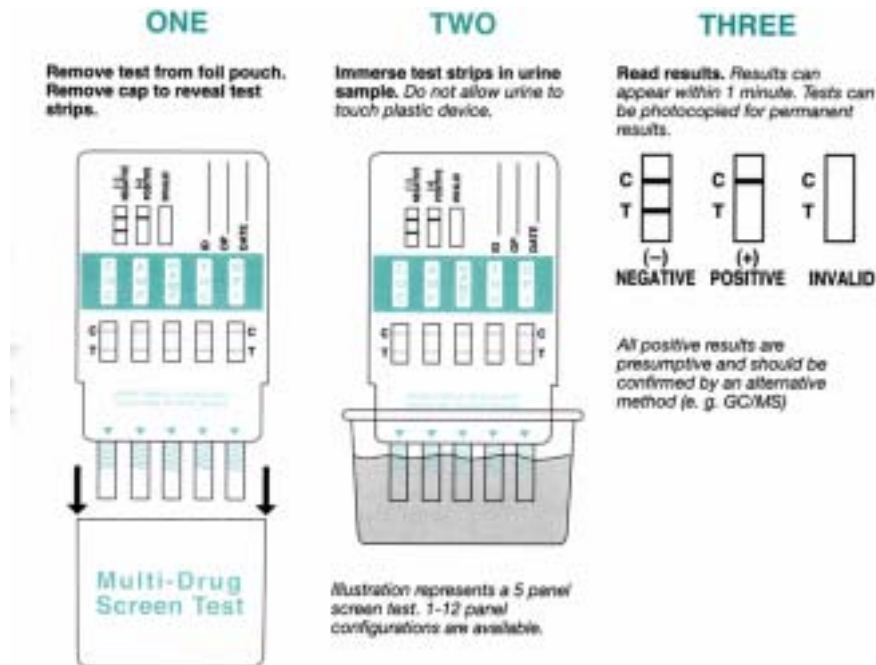

1. Bring the pouch to room temperature before opening it. Remove the test card from the sealed pouch and use it as soon as possible. Remove the cap from the end of the test card. With the arrows pointing toward the urine specimen, immerse the strip(s) of the test card vertically in the urine specimen for at least 10-15 seconds. Immerse the test card to at least the level of the wavy lines on the strip(s), but not above the arrow(s) on the test card.
2. Place the test card on a non-absorbent flat surface, start your timer and wait for the red line(s) to appear. The results should be read at 5 minutes. Do not interpret results after 10 minutes.

### INTERPRETATION OF RESULTS

**NEGATIVE:** Two lines appear. One red line should be in the control region (C), and another apparent red or pink line adjacent should be in the test region (T). The negative indicates that the drug concentration is below the detectable level.

**\*NOTE:** The shade of red in the test line region (T) will vary, but it should be considered negative whenever there is even a faint pink line.

**POSITIVE:** One red line appears in the control region (C). No line appears in the test region (T). This positive result indicates that the drug concentration is above the detectable level.

**INVALID:** Control line fails to appear. Insufficient specimen volume or incorrect procedural techniques are the most likely reasons for the control line failure. Review the procedure and repeat the test using a new test panel.

**WWW.DRUGFORMATION.COM**  
**(888) 520-6690**